




## Application guide & folding techniques


Can be used as a primary or secondary dressing • Suitable for use under compression

- 1 Cleanse the wound according to local procedures
- 2 Select an appropriate dressing size. The dressing pad needs to overlap the edge of the wound. Do not cut the dressing
- 3 Open the pack and lay the sterile pad on the wound. Either side of the sterile pad may be used. You may stack dressings for additional absorption
- 4 Cover with a bandage or wrap to secure the dressing, refer to your local guidelines for further guidance
- 5 Check the dressing at regular intervals and change according to the clinical condition of the wound, or when saturated

### In-between toes/fingers

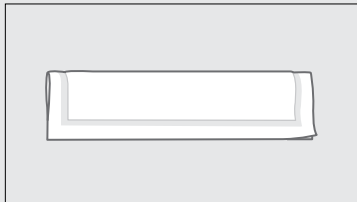


- 1 Fold a 4 × 9" in half lengthways

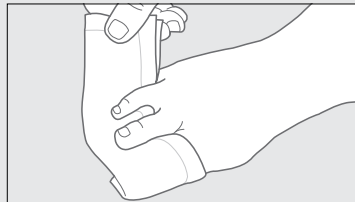


- 2 Weave between toes/fingers




### Heel/elbow

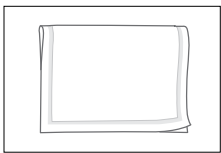


- 1 Fold a 8 × 9" in half

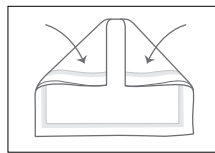


- 2 Fold each corner inwards

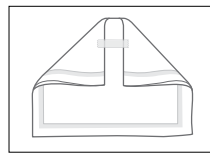

- 3 Place tape to hold folded corners in place

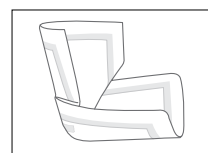


- 4 Open out folded dressing


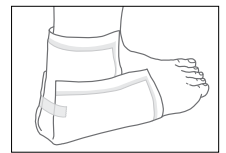


- 5 Place on heel/elbow


## How to apply KERRAMAX CARE™ Multisite Dressing:

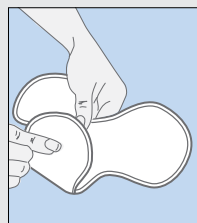


Breast

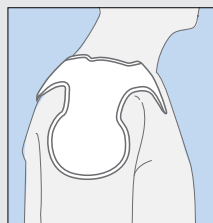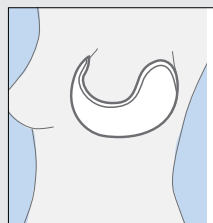


Shoulder

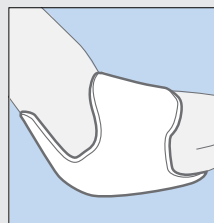

Elbow

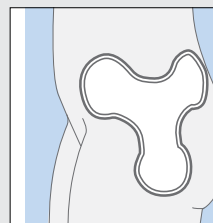


Hip

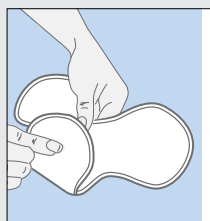

Groin



Heel

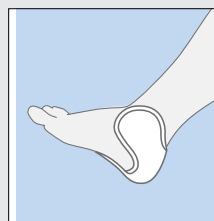

KERRAMAX CARE™ Dressing is a super-absorbent primary or secondary dressing, suitable for chronic and acute wounds, designed to help improve wound healing.

Product	Size	Surface area, in <sup>2</sup>	Dressings per box	HCPCS
PRD500-025	2 x 2 in	4	10	A6196
PRD500-050	4 x 4 in	16	10	A6196
PRD500-100	5 x 6 in	30	10	A6197
PRD500-120	4 x 9 in	36	10	A6197
PRD500-240	8 x 9 in	72	10	A6197
PRD500-380	8 x 12 in	96	5	A6198
PRD500-600	8 x 20 in	160	5	A6198
PRD500-300	8 x 9 in Multisite	29	5	A6197

For more information, call 800-275-4524 or visit [myKCI.com](http://myKCI.com)

**NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. Rx only.**

© 2020 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. PRA-PM-US-02799 (10/20)

