




Dehiscent Surgical Wound

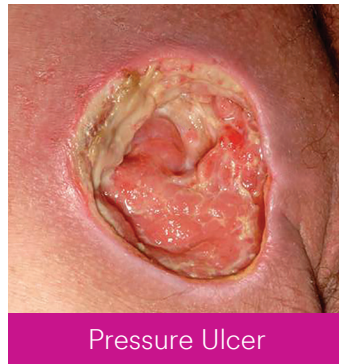

Pressure Ulcer



Diabetic Foot Ulcer



Traumatic Wound



Venous Leg Ulcer



Acute Wound



Chronic Wound



Infected Wound†

† When used in conjunction with good clinical practice; utilize appropriate debridement and/or antibiotics where necessary.
Untreated osteomyelitis is contraindicated for use with 3M™ V.A.C.® Therapy.

NOTE: This form is for use by Wound Care Centers, Physician Offices, Home Health Agencies, Outpatient Centers, and Skilled Nursing Facilities staff to assist in evaluating patients. This form is not a medical prescription and is not a guarantee of product approval or reimbursement. Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product Instructions for Use prior to application. Rx only.

Images above are not to scale.

Healing starts here

Please Answer **YES** or **NO** to These 2 Questions:

1 ☐ **YES** ☐ **NO**

Is the wound etiology one of the following:

Chronic, acute, traumatic, subacute and dehiscent, partial-thickness burns*, ulcer (such as diabetic, pressure or venous insufficiency), flap or graft?

* 3M™ Snap™ Therapy System is not indicated for burns.

2 ☐ **YES** ☐ **NO**

Is the wound free of eschar?



If Both Answers Are YES,


see reverse to determine if the 3M™ ActiV.A.C.™ Therapy System or the 3M™ Snap™ System may be a treatment option for your patient

If Any Answers Are NO,

see reverse for information on 3M Advanced Wound Care & Skin Integrity Solutions

Select the appropriate product

Wound/Clinical Considerations (Non-Hospital)		
CRITERIA	3M™ ActiV.A.C.™ Therapy System	3M™ SNAP™ Therapy System
Wound Size	> 13cm x 13cm	≤ 13cm x 13cm
Wound Drainage	> 180mL/week	≤ 180mL/week (3) x 60mL cartridges/week
NPWT Initiation	Previously treated with Modern Wound Care; > Day 30	First line treatment; Day 1* (No 30 day requirement for modern wound care, No minimum depth) *may vary by payor

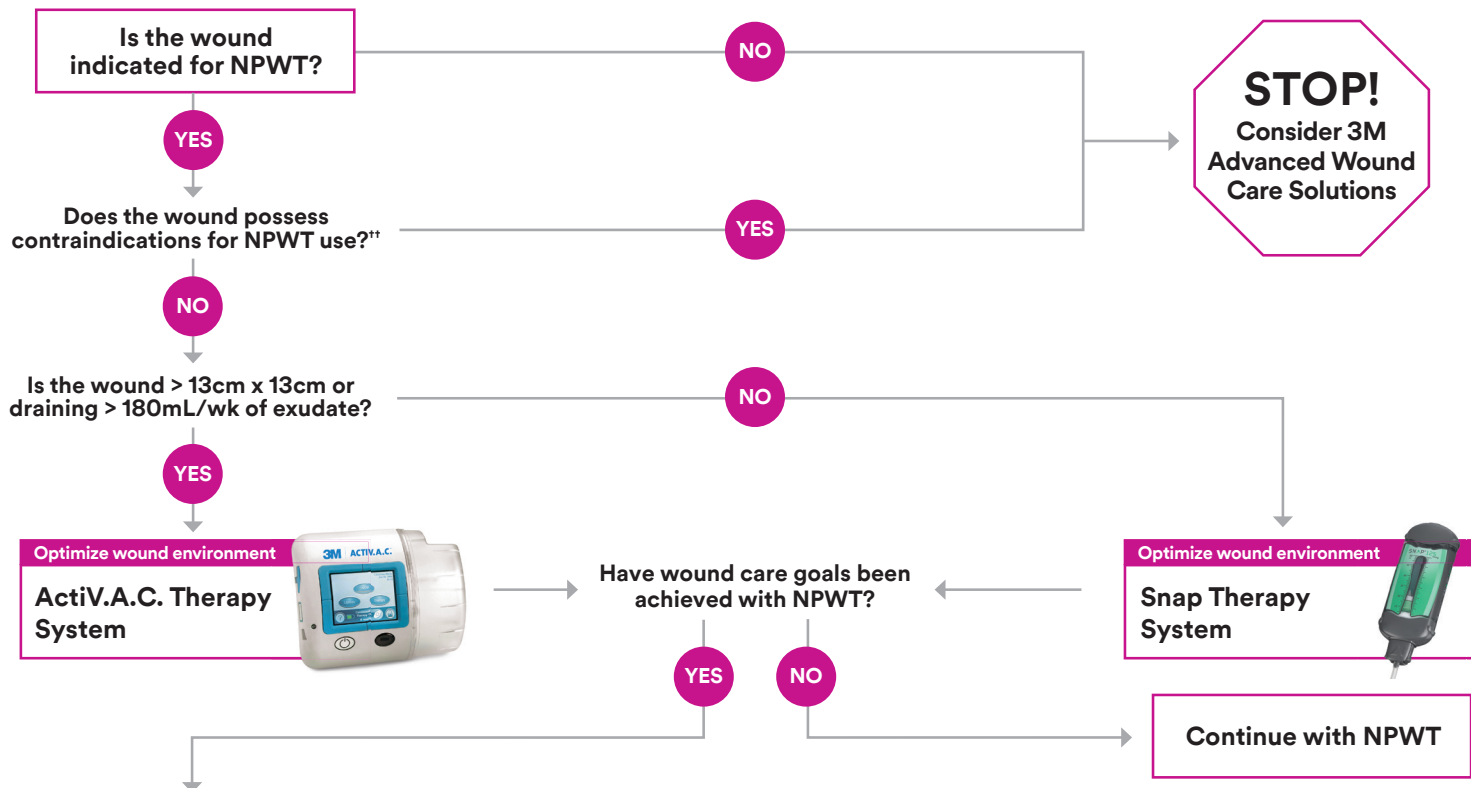

When the time is right to transition from NPWT, consider 3M Advanced Wound Care & Skin Integrity Solutions

3M Advanced Wound Care Solutions may be appropriate when NPWT is not available or appropriate and should be selected based on clinical goal.

Protect skin	Protect/Prepare wound bed		Optimize wound environment		Provide therapeutic compression
Routine skin protection	Manage biofilm/bioburden		Manage exudate		
3M™ Cavilon™ No Sting Barrier Film	3M™ Kerracel™ Ag Dressing with Ag OXY-SALTS™ Technology	3M™ Silvercel™ Non Adherent Dressing	3M™ Kerracel™ Gelling Fiber Dressing	3M™ Kerramax Care™ Super-Absorbent Dressing	3M™ Coban™ 2 Two-Layer Compression Systems
At-risk or damaged skin protection	Provide collagen	Minimize adherence	3M™ Tegaderm™ Silicone Foam Dressings	3M™ Tegaderm™ High Performance Foam Dressings	3M™ Coban™ 2 Lite Two-Layer Compression Systems
3M™ Cavilon™ Advanced Skin Protectant	3M™ Promogran Prisma™ Matrix with ORC Technology	3M™ Adaptic™ Touch Non-Adhering Silicone Dressing			



3M Company
3M Center, Building 275-4E-01
St. Paul, MN 55144 USA

Phone 1-800-228-3957
Web 3M.com/Medical

Contact your 3M Account Representative for more information.
Call **800-275-4524** or visit **3M.com/KCI**

* NOTE: Specific instructions, contraindications, warnings, precautions, and safety information exist for these products and therapies, some of which may be Rx only. Please consult a clinician and product instructions for use prior to application.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited.
Used under license in Canada. Ag OXY-SALTS is a trademark of Argent LP. PRA-PM-US-02092 (03/21) US_70-2011-8038-0