



*Puracyn<sup>®</sup> Plus Wound Irrigation Solution is a next-generation wound irrigation and management solution designed to improve the most essential part of the wound treatment process: preparation of the wound bed. This advanced pH-balanced solution is used in conjunction with wound irrigation, debridement, and dressing procedures, which set the stage for optimum healing conditions.*


## PURACYN<sup>®</sup> PLUS BENEFITS

The Wound Irrigation Solution is a FDA 510(k) cleared product, and has many benefits:

- Non-toxic and non-irritating
- Contains no steroids or antibiotics
- Contains no alcohol or iodine
- No mixing or diluting required
- Up to 24 month shelf life
- pH balanced (6.2-7.2)
- Supports the reduction of wound odors
- Irrigates the wound bed without harming healthy tissue
- Environmentally friendly—no special disposal required

## ADVANCED HYPOCHLOROUS TECHNOLOGY

The hypochlorous molecule is naturally produced in the body's neutrophils.<sup>1</sup> Puracyn Plus is a synthesized version of the hypochlorous molecule to provide clinicians an alternative to commercial wound cleansers that may inhibit wound closure.

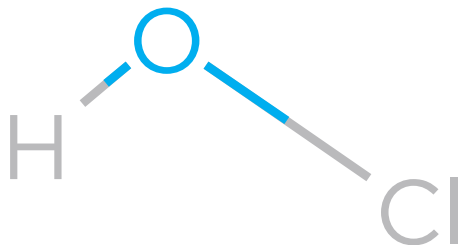

## INDICATIONS FOR USE

The Wound Irrigation Solution is intended for use by healthcare professionals in the cleansing, irrigating, moistening, and debridement of the following types of wounds:

- Acute and chronic wounds
- Partial- and full-thickness wounds
- First- and second-degree burns
- Neuropathic/Diabetic ulcers
- Stasis ulcers
- Pressure injuries (stage I-IV)
- Post-surgical wounds
- Grafted and donor sites
- Abrasions and minor skin irritations
- Trauma wounds

## PURACYN<sup>®</sup> PLUS SAFETY

There are no known drug/treatment contraindications with Puracyn Plus. Additional testing has demonstrated no oral or ocular toxicity.

## APPLICATION RECOMMENDATIONS

- Cleanse and irrigate wound beds to help remove debris and other contaminants
- Saturate dressings prior to application
- Based on levels of bioburden and to reduce malodor, soak wounds for up to 3-5 minutes
- Available in spikeable configuration that is compatible with various types of debridement procedures, such as Negative Pressure Wound Therapy, ultrasonic debridement, hydrosurgery debridement, and pulse lavage
- Can be used to moisten encrusted dressings to help facilitate removal

1. Thomas EL, Lehrer RI, Rest RF. Human Neutrophil antimicrobial activity. Rev Infect Dis 1988; 10 Suppl 2: S450-6.



## Product Order Guide

UPC	Item #	Description	Size	Applicator	HCPCS Code	Case Quantity
818582010856	6502	Wound Irrigation & Management Solution	55 ml (2 oz)	Twist Top	A6260	12
818582010993	6503	Wound Irrigation & Management Solution	55 ml (2 oz)	Pump Spray	A6260	12
818582010764	6504	Wound Irrigation & Management Solution	120 ml (4 oz)	Twist Top	A6260	6
818582011006	6505	Wound Irrigation & Management Solution	120 ml (4 oz)	Pump Spray	A6260	6
818582010788	6508	Wound Irrigation & Management Solution	250 ml (8.5 oz)	Twist Top	A6260	6
818582011013	6509	Wound Irrigation & Management Solution	250 ml (8.5 oz)	Pump Spray	A6260	6
818582010771	6516	Wound Irrigation & Management Solution	500 ml (16.9 oz)	Septum Cap	A6260	6
818582011020	6517	Wound Irrigation & Management Solution	500 ml (16.9 oz)	Trigger	A6260	6
818582011723	6518	Wound Irrigation & Management Solution	1000 ml (32 oz)	Septum Cap	A6260	6
818582011327	6525	Wound Irrigation & Management Solution	475 ml (16 oz)	Instill/Spikeable	A6260	6
818582011334	6527	Wound Irrigation & Management Solution	1000 ml (32 oz)	Instill/Spikeable	A6260	6
818582011808	6542	Antimicrobial Hydrogel	90 ml (3 oz)	Twist Cap	A6248	6

Puracyn Plus products contain hypochlorous acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms within the product.

### TO ORDER, OR FOR MORE INFORMATION ON PURACYN PLUS:

**Customer Service:** 1.866.PURACYN  
1.866.787.2296 (8am PST - 5pm PST)

**Fax/order:** 1.909.428.1947

**Address:** 3456 N. Riverside Avenue, Rialto, CA 92377

**Email/Product Info:** puracynpluspro@innovacyn.com

